



1

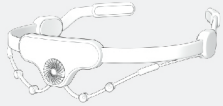

Position the patient's head for easy and stable access to the occiput. For example, in a seated position, have the patient place their hands on a flat surface and then rest their forehead on their hands.


Remove the OcciGuide from the packaging.

2

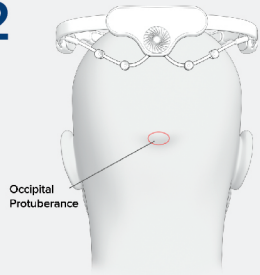


Locate the patient's occipital protuberance and place the device on the patient's head with the OcciGuide Logo resting directly over the occipital protuberance.

3

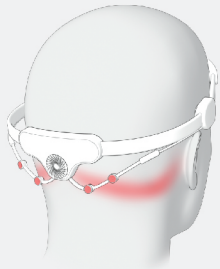


With antiseptic, thoroughly cleanse all four (4) injection ports on the device as well as the hair and skin along the pathway of the occipital nerves between the mastoid process and occipital protuberance.

4

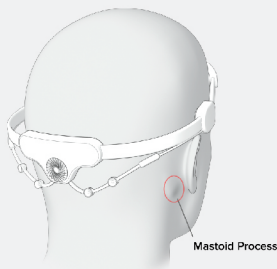


Adjust the metal locator arm down from the headband until the end rests on the mastoid process. This will align the device injection ports along the pathways of the greater and lesser occipital nerves on the occiput. This can be done on one or both sides of the head.

5

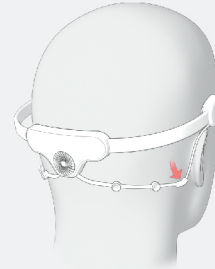


The injection ports must be over the occiput prior to injection. When the device is placed correctly, insert the needle through the injection port. The needle should be tilted slightly upward as it is advanced to the occiput directly beneath the injection port.

6

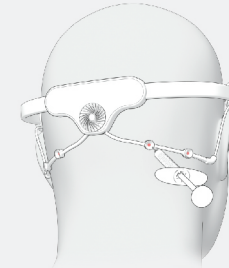


A 1 to 1.5 inch needle can be advanced up to its full length or until contact is made with the bone. The needle is then slightly withdrawn prior to injecting. Proper aspiration techniques should be utilized prior to expressing the solution to ensure no vascular uptake occurs such as into the occipital artery.

7

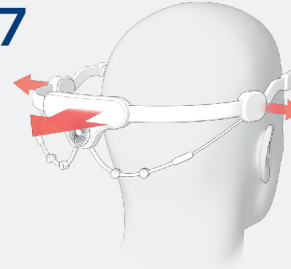

Once the injections have been completed, place a hand on each side of the headband and remove slowly, pulling away from the patient's head.

8

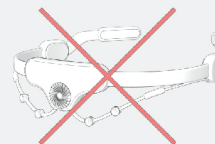

After use, dispose of the device in accordance with clinical protocols.

**Warning:** This is a single use device for multiple reasons, including loss of accuracy after single use, contamination with blood products, etc.